

IDI-Strep B™

Vaginal/Rectal Specimen Collection Protocol


1



Before collecting the specimen, **WIPE AWAY** excessive amount of secretion or discharge from the vaginal area.


A Copan Venturi Transystem® (141C) must then be used to collect the vaginal/rectal specimen

2

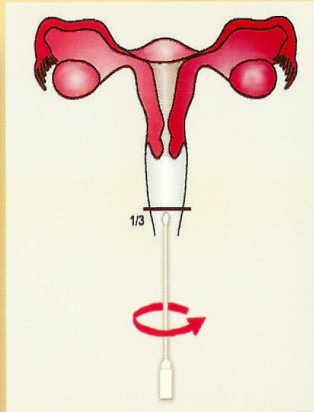


Insert the swab into the vagina and sample secretions from mucosa of the lower one-third of the vagina, then...

3

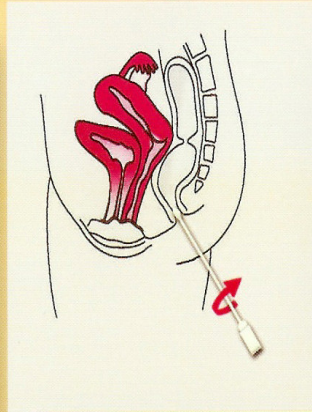

Insert the same swab approximately 2.5 cm beyond the anal sphincter and gently rotate

4

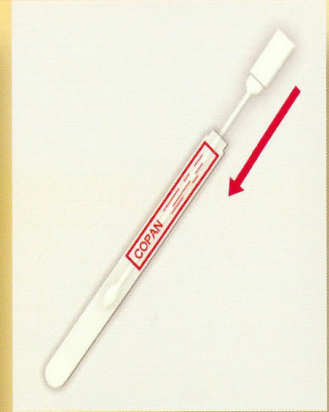

Return swab to its container and send to laboratory immediately



Cepheid

Technical Support:
1-888-838-3222